

FFP2 NR
Protective
Fold Flat Mask
A1M

The Purple's

16*14*15cm

The Purple's



FFP2 NR
Protective Fold Flat Mask
A1M
PFE > 95%



25 PCS

INDIVIDUALLY PACKED

5-layer protection

2-layer melt-blown Fabric

Anti-Fog foam nose bridge

PPE-Regulation(EU)2016/425
EN 149:2001+A1:2009 FFP2 NR

Single-use face mask (Non-sterile)

Indications For Use

It is used for patients in order to reduce the risk of the spread of infection, and it is intended for use by healthcare professionals in an operating room or in other medical settings with similar requirements.

Limitations

- Not suitable for working places where open flames exist (such as welding, casting, etc).
- It is not suitable for professional protective use such as oxygen deficient, toxic gas environment.
- For healthcare use.

Checks prior to use

Please check whether the packaging is damaged, whether the mask body is damaged, deformed or has other obvious defects before use.

Fitting

1. Cup the mask in one hand.
2. Completely cover your nose and mouth with the mask.
3. Place the ear straps around both ears.
4. For optimal hold adjust the metal nose clip to your nose.
5. Cover the mask with both hands and exhale. No air should escape from the side.



If transporting this product use original packaging provided

The Purple's



FFP2 NR
Protective Fold Flat Mask
A1M
PFE > 95%



25 PCS

INDIVIDUALLY PACKED

5-layer protection

2-layer melt-blown Fabric

Anti-Fog foam nose bridge

PPE-Regulation(EU)2016/425
EN 149:2001+A1:2009 FFP2 NR

Use cautions:

- Please check fit of mask prior to use;
- It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal;
- Please do not use the mask under low air quality condition, such as serious contaminants in air, oxygen deficiency;
- Do not use the mask in explosive atmosphere;
- After use, please discard the mask following local regulation;
- The mask is only for single use, do not use repeatedly.


Storage Condition:

The product should be stored in a well ventilated, dark and dry environment. Keep away from fire, pollutants and possible pollution sources. Also, the transport storage 's temperature: -20 C ~38 C, relative humidity: <80%RH.

Shelf-Life:

The Protective Mask is valid for 3 years with the above storage condition.

Website: www.thepurplesmask.com sales@thepurplesmask.com
Manufacturer: Zhejiang The Purples Protective Products Co.,Ltd.
Address: No. 45, Suhua Street, Suxi Town, Yiwu, Zhejiang Province, China

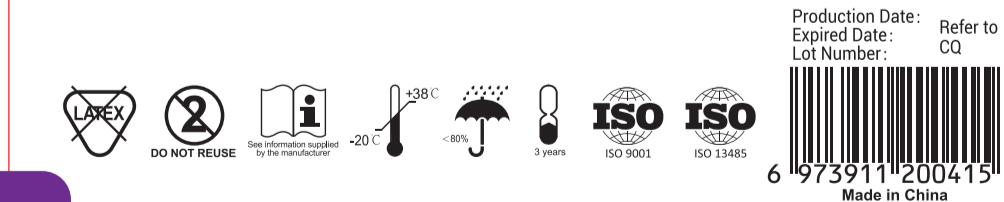

Production Date: Refer to Q3
Expired Date:
Lot Number:
6 973911 200415
Made in China